





intelligent orthopaedics

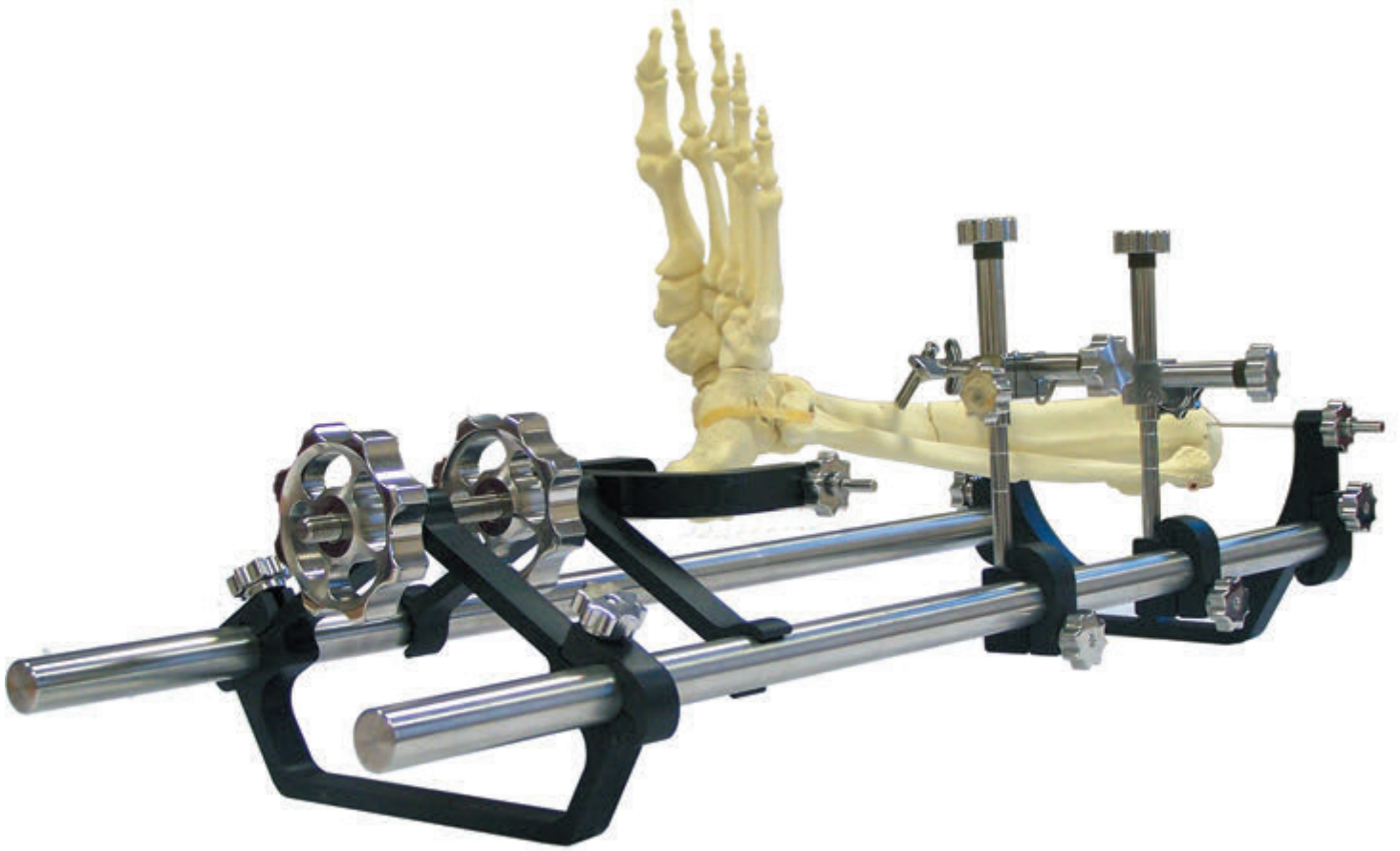


Quick Assembly Guides

Quick assembly instructions

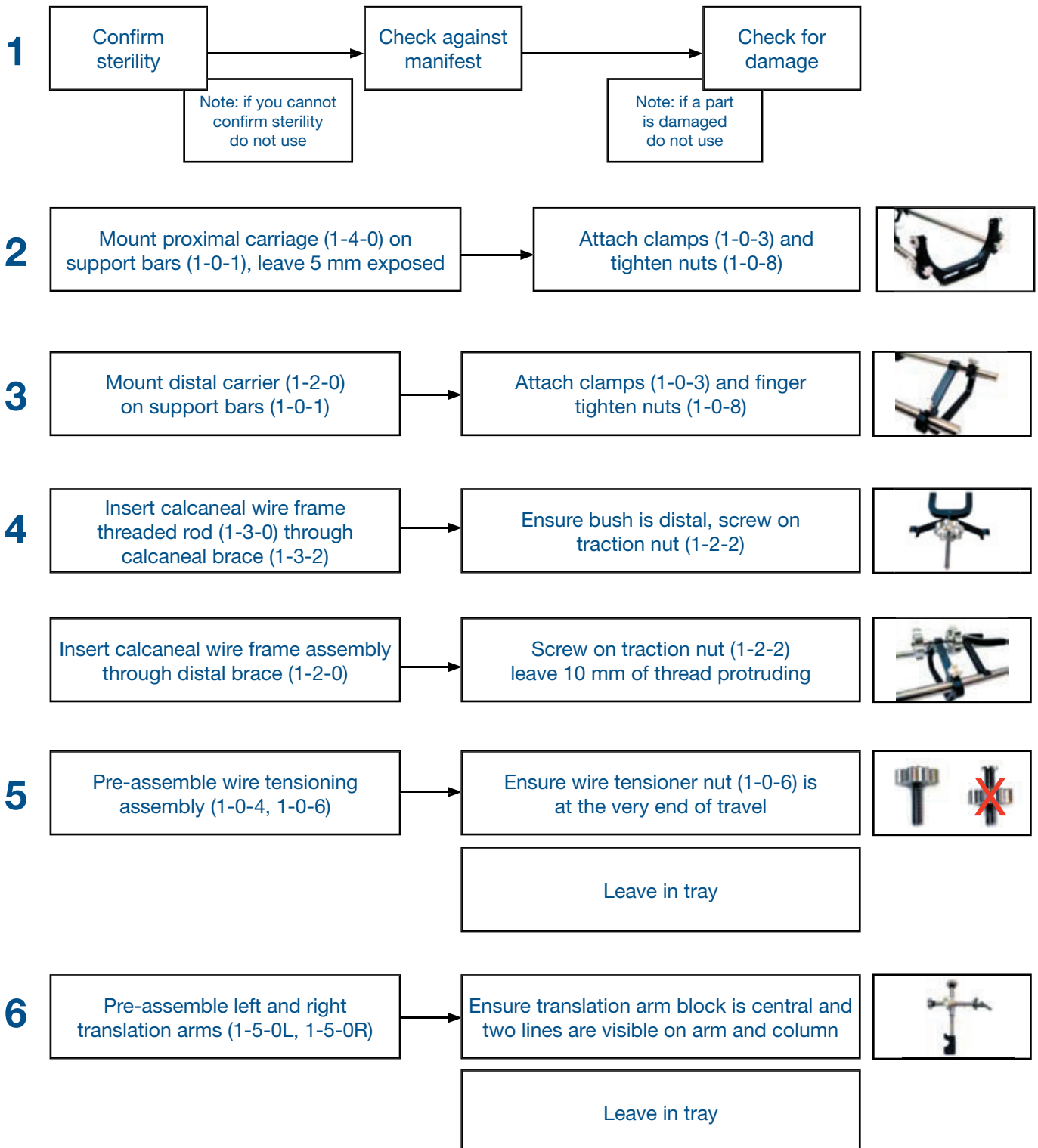


Quick disassembly instructions

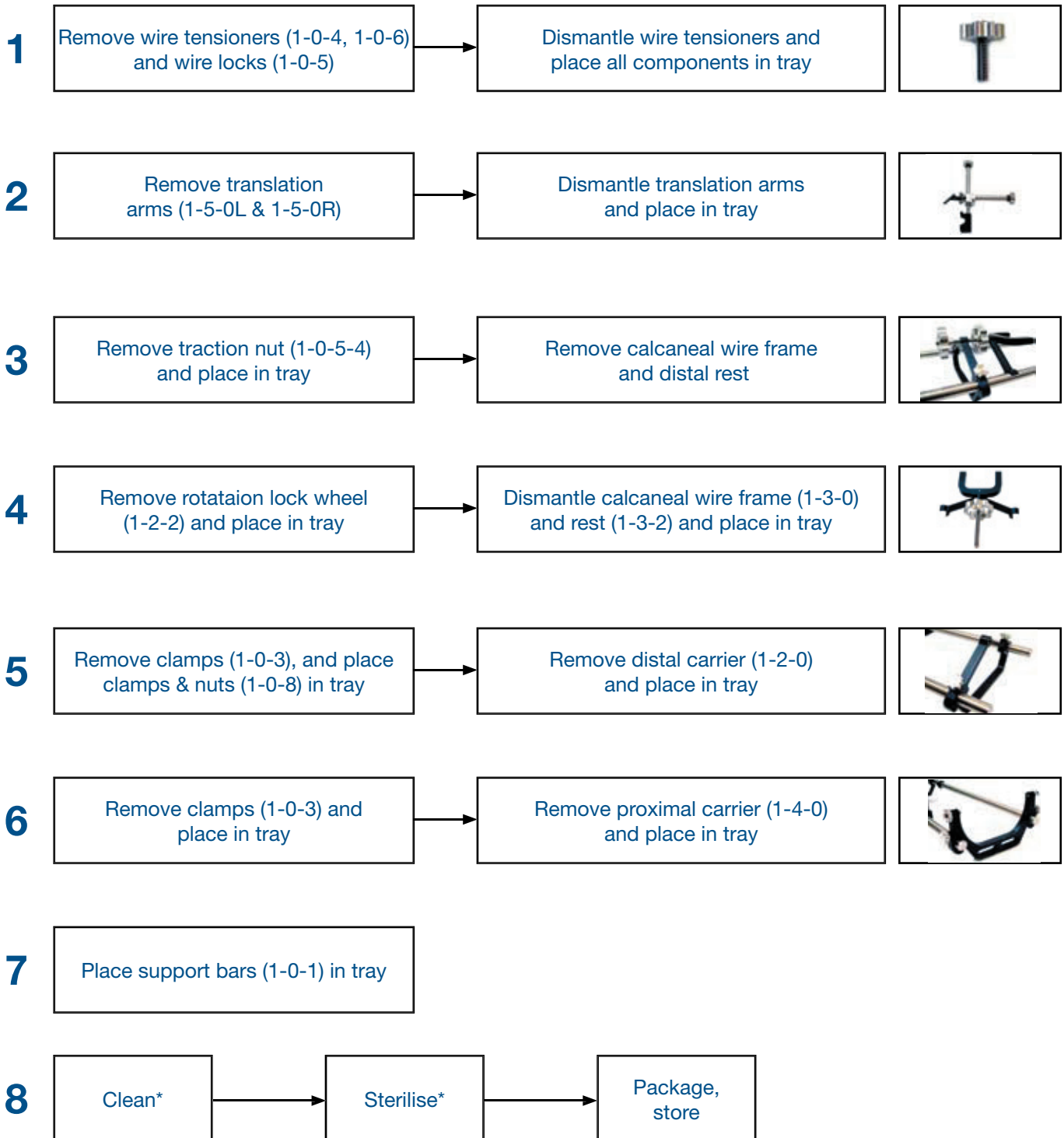


* See cleaning and sterilisation instructions

Quick assembly instructions (translation arms)

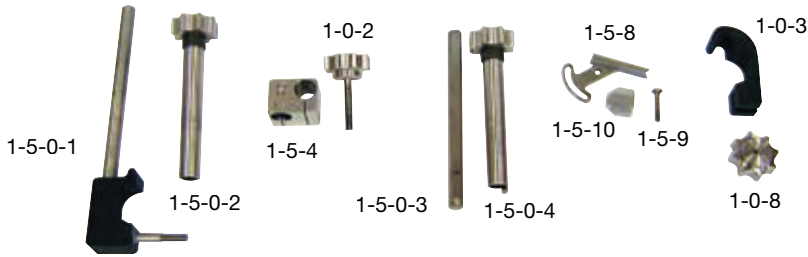

1

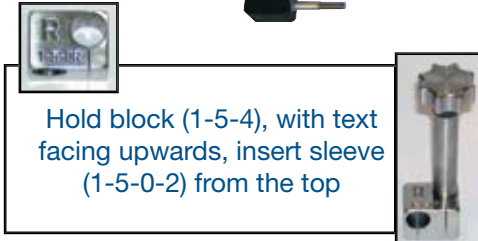


Hold block (1-5-4), with text facing upwards, insert sleeve (1-5-0-2) from the top

Screw column (1-5-0-1) into (1-5-0-2) up to middle line

Note: You will need to push slightly to engage



2

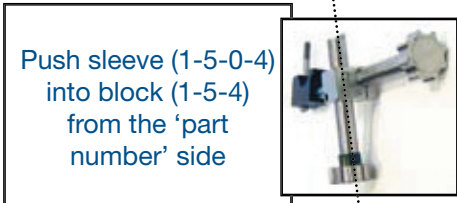


Push sleeve (1-5-0-4) into block (1-5-4) from the 'part number' side


Screw arm (1-5-0-3) into (1-5-0-4) up to middle line

Note: You will need to push slightly to engage

Insert screw (1-0-2) from opposite side of text, do not tighten

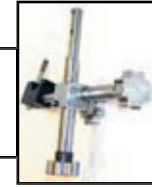


3

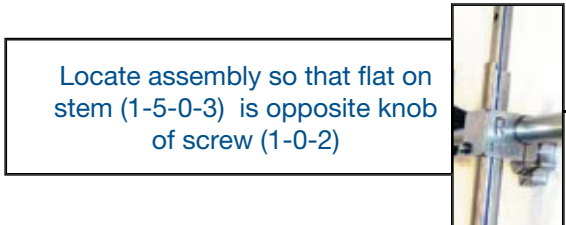


Locate assembly so that flat on stem (1-5-0-3) is opposite knob of screw (1-0-2)

Ensure that block (1-5-4) is central and tighten screw (1-0-2)

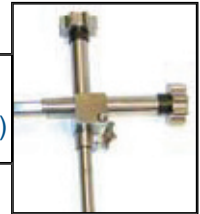

4

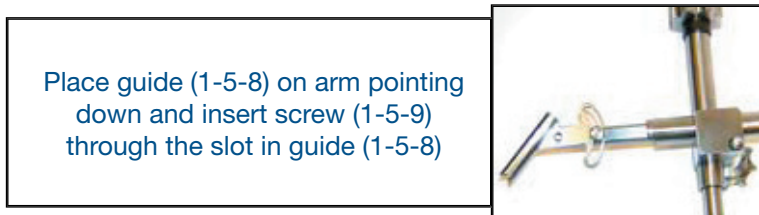


Place guide (1-5-8) on arm pointing down and insert screw (1-5-9) through the slot in guide (1-5-8)

Screw nut (1-5-10) onto screw (1-5-9), tighten lightly with guide at 45 degrees

5



Leave nut (1-0-8) and clamp (1-0-3) in tray

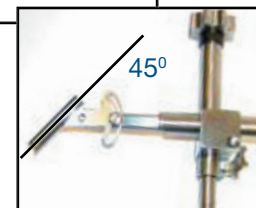

For more information contact

Intelligent Orthopaedics Ltd
PO Box 2281
Stoke on Trent
UK ST4 9BE

E: www.io-uk.com
T: +44 (0) 844 800 4405